



# STARMed Ventukit

Hood for CPAP therapy with built-in Venturi flow driver



Critical Care ▪ Patient Interfaces ▪ **STARMed**

# STARMed Ventukit CPAP hood with Venturi flow driver


## StarMed Ventukit

StarMed Ventukit is a complete and easy to use system simply activated by connecting to the O<sub>2</sub> source available in the hospital departments. StarMed Ventukit unique design consists of oxygen tubes and a Venturi flow driver integral to a CPAP hood. The kit also includes an adjustable PEEP valve for immediate therapy activation.

Read more evidence on [www.intersurgical.com](http://www.intersurgical.com)

## Technical Features

- The integrated Venturi flow driver can generate high patient flows while consuming a reduced amount of compressed oxygen
- O<sub>2</sub>% can be adjusted from 40% to 100% at each PEEP level
- The adjustable PEEP valve can provide patient pressures in a range of settings up to 20 cmH<sub>2</sub>O
- The user is guided by a performance table integral to the product, for independent PEEP and O<sub>2</sub>% settings

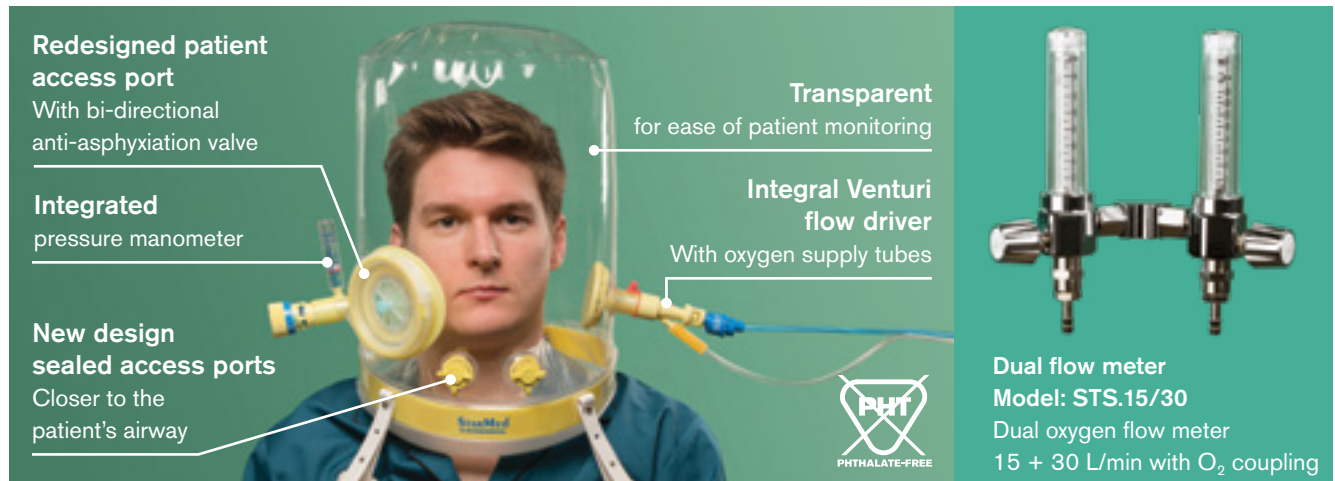

## Ventukit CPAP hood with Venturi flow driver, bi-directional anti-asphyxiation valve and patient access port, adjustable PEEP valve and ear plugs

Performance table subject to changes. Always refer to product labelling.

Requirement		Setting	Flow delivered
O <sub>2</sub> % approx	PEEP cmH <sub>2</sub> O	Flow O <sub>2</sub> L/min A + B	Flow approx L/min
40%	5	9 + 0	46
	7.5	9 + 0	47
	10	10 + 0	48
	12.5	10 + 0	46
	15	11 + 0	47
	20	12 + 0	48
50%	5	8 + 9	47
	7.5	8 + 9	47
	10	9 + 9	45
	12.5	9 + 8	48
	15	10 + 8	50
	20	12 + 9	54

Requirement		Setting	Flow delivered
O <sub>2</sub> % approx	PEEP cmH <sub>2</sub> O	Flow O <sub>2</sub> L/min A + B	Flow approx L/min
60%	5	7 + 16	52
	7.5	8 + 16	48
	10	8 + 14	48
	12.5	9 + 13	47
	15	9 + 13	47
	20	10 + 14	47
100%	Feed with a minimum O <sub>2</sub> flow of 40 L/min		

Code	Size	Neck circumference	Box Qty.
CV100XXL/2	Extra Extra Large	≥50cm	5
CV100XL/2	Extra Large	45-52cm	5
CV100L/2	Large	40-47cm	5
CV100M/2	Medium	34-41cm	5
CV100S/2	Small	27-34cm	5
CV100XS/2	Extra Small	17-27cm	5



[www.intersurgical.com/info/starmed](http://www.intersurgical.com/info/starmed)

Make an enquiry

Dual flow meter manufactured by FLOW METER S.p.A.



Intersurgical Ltd, Crane House, Molly Millars Lane, Wokingham, Berkshire, RG41 2RZ, UK  
 T: +44 (0)118 965 6300 F: +44 (0)118 965 6356 info@intersurgical.com www.intersurgical.com



Manufacturer: Intersurgical S.p.A. - Mirandola (MO) - Italy  
 The Intersurgical S.p.A. Organisation has a Quality Management System Certificate by KIWA CERMET ITALY S.p.A. according to ISO 9001:2015 and ISO 13485:2016

**Please think before you print**  
 Save energy and paper. If you must print this information sheet please print it double sided.

IS1.21\_INT • Issue 7 02.19