



# STARMed Ventumask Basic

For CPAP therapy with Venturi flow driver and adjustable PEEP valve



Critical Care ▪ Patient Interfaces ▪ **STARMed**

# STARMed Ventumask Basic with Venturi flow driver & adjustable PEEP valve

## StarMed Ventumask Basic

StarMed Ventumask Basic is a unique design for CPAP therapy that is simply activated by connecting to an appropriate oxygen supply. It can be used in a hospital environment or pre-hospital use, for example in an ambulance.

### Advantages and Technical Features

- Ventumask Basic is a complete and easy to use system, where the oxygen supply tube, a Venturi flow driver and a PEEP valve are integral to the CPAP mask
- The integrated Venturi flow driver is operated by a single oxygen flowmeter to generate high patient flows while consuming a reduced amount of compressed gas
- The built-in adjustable PEEP valve is designed exclusively for Ventumask Basic and the administration to the patient is facilitated at some pre-settings: 5 - 7.5 - 10 - 12.5 cmH<sub>2</sub>O
- O<sub>2</sub>% of 40% can be delivered at each PEEP level
- The user is guided by a performance table integral to the product, to be aware of patient flow at each PEEP settings

**The kit is composed of:**

- CPAP mask with safety valve and port to monitor actual O<sub>2</sub>% or pressure
- Venturi flow driver integral to the mask
- Adjustable headgear included
- Adjustable PEEP valve designed for Ventumask Basic facilitates some pre-calibrated pressure settings
- Integral oxygen tube

## Ventumask Basic for CPAP therapy with Venturi flow driver and adjustable PEEP valve

Requirement		Setting	Flow delivered
O <sub>2</sub> % approx	PEEP cmH <sub>2</sub> O	Flow O <sub>2</sub> L/min	Flow approx L/min
40%	5	12	57
	7.5	12	52
	10	12	47
	12.5	12	42

Performance table subject to changes. Always refer to product labelling.

Code	Description	Size	Box Qty.
CV203S	Ventumask Basic CPAP mask with integral Venturi flow driver and adjustable PEEP valve	Small	5
CV203M/L	Ventumask Basic CPAP mask with integral Venturi flow driver and adjustable PEEP valve	Medium / Large	5
CV203XL	Ventumask Basic CPAP mask with integral Venturi flow driver and adjustable PEEP valve	Extra Large	5



[www.intersurgical.com/info/starmed](http://www.intersurgical.com/info/starmed)

Video available at [www.intersurgical.com](http://www.intersurgical.com)

Make an enquiry



Intersurgical Ltd, Crane House, Molly Millars Lane, Wokingham, Berkshire, RG41 2RZ, UK  
 T: +44 (0)118 965 6300 F: +44 (0)118 965 6356 info@intersurgical.com www.intersurgical.com



Manufacturer: Intersurgical S.p.A. - Mirandola (MO) - Italy  
 The Intersurgical S.p.A. Organisation has a Quality Management System Certificate by  
 KIWA CERMET ITALY S.p.A. according to ISO 9001:2015 and ISO 13485:2016

**Please think before you print**  
 Save energy and paper. If you must print this information sheet please print it double sided.

IS1.45\_INT • Issue 4 02.19